

# Venimex Extractor



- Effective against bites and stings. Prevents poison from spreading through out the body.
- Automatic vacuum pump with strong suction power of 800millibars.
- Includes 2 suction caps (800 millibars when using the small suction cup).



## Instruction for Use

Use the venimex to remove venom from an sting or bite. This prevents the venom from spreading throughout the body. The automatic vacuum pump with a strong suction power (800 mbar) is easy to operate with one hand. The suction cups can be used in three ways. Place the venom extractor over the bite area and press it down firmly until the pump unlocks. Allow the automatic vacuum pump to suck between one and five minutes. The venimex is suitable for all ages. It is intended as a first aid measure. We recommend that you read the instruction leaflet carefully and test the Venimex before hand in order to use it more efficiently and faster.



### Step 1:

Use the venom extractor after you have been stung or bitten.




### Step 5:

Allow the automatic vacuum pump to suck for one minute (five minutes in the case of a scorpion sting)



### Step 2:

Select an appropriately sized suction cup. Use the small suction cup for uneven body parts (e.g. your knee or finger). The small suction cup can be attached to the pump in two ways.

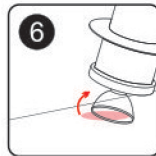

### Step 6:

Remove the pump by tilting it until air flows into the suction cup. Disinfect the wound with an alcohol-free antiseptic.



### Step 3:

Place the suction cup over the bite area.




### Step 7:

Clean the suction cup with soap and water or an antibacterial wipe. The pump itself must not be rinsed.



### Step 4:

Press the venom extractor down firmly until the pump unlocks.

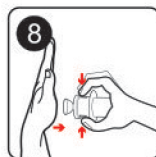

### Step 8:

For storage, push the inner tube into the pump and press the tabs on both sides to lock it. Store the pump in its protective pouch in a dry place at room temperature.

## Safety Recommendations

- Remain calm after being bitten or stung.
- Anxiety circulates venom through the body and reduces the effectiveness of the venimex.
- The venimex is not a replacement for medical assistance in serious cases.
- Do not use on mucous membranes, eyelids and genitals.
- If you have been stung by a bee, do not remove the sting when using the Venimex.
- In the case of allergic reactions, the Venimex cannot prevent angioedema.
- Keep out of reach of children.