

# ChitoSAM™100

**SAM**<sup>®</sup>  
MEDICAL

100% CHITOSAN HEMOSTATIC DRESSING

FOR TEMPORARY EXTERNAL USE ONLY

For the control of moderate to severe bleeding and management of external abrasions, lacerations.

## INSTRUCTIONS FOR USE

1. Use appropriate size and # of dressings based on wound area and amount of bleeding.
2. Apply dressing directly to the wound.
3. To control bleeding, apply firm pressure for 3 minutes. Secure with adequate cover dressing, maintaining pressure to the wound.

### CAUTION/WARNING:

Do not place over eyes.  
Contains chitosan from shellfish.

Use sterile saline or water to rinse any dressing remnants from the wound when removing.

### WARNING:

**If shellfish allergy is a concern, please consult your doctor.  
Dressing must be removed within 24 hours.**

- Not made with natural rubber latex
- Do not resterilize
- Do not reuse
- Do not use if package is damaged or open

<b>REF</b>	<b>CT101-A-EN</b> / 4"x4" (10 cm x 10 cm)   Single Ply Hemostatic Dressing
<b>REF</b>	<b>CT101-B-EN</b> / 3"x4" (7.6 cm x 1.22 m)   Z fold Hemostatic Dressing
<b>REF</b>	<b>CT101-C-EN</b> / 3"x6" (7.6 cm x 1.83 m)   Z fold Hemostatic Dressing

MADE IN TAIWAN

Manufactured by  
BenQ Materials Corporation  
29, Jianguo E. Road, Gueishan Taoyuan, Taiwan, R.O.C.  
+886.3.374.8800  
benqmaterials.com

**CE** 2460

**EC REP**

Obelis S.A, Bd. Général Wahis, 53,  
1030 Brussels, Belgium

Distributed By:  
SAM Medical Products®  
12200 SW Tualatin Rd, Ste 200  
Tualatin, OR 97062 USA  
+1 503.639.5474

[sammedical.com](http://sammedical.com)

## SYMBOL GLOSSARY

Manufacturer	
Batch Code	
Catalog Number	
Do Not Reuse	
Do not use if package is damaged	
Do not resterilize	
Not made with natural rubber latex	
Temperature range	
Sterilized using irradiation	
Use by date	
Keep dry	
Authorized Representative in the European Community	
Medical Device	
Keep away from sunlight	
CE Mark	
Importer	

CT-101-EN-1  
ISSUE DATE: 2021-06