WNL
PRODUCTS
Practi-TRAINER
Universal AED Trainer

- 1** TURN ON
- 2** FOLLOW PROMPTS
- 3** PRESS SHOCK BUTTON IF INSTRUCTED

WL 220 | AED Trainer

English/Spanish w/ Remote

Directions for Use

CE
Copyright 2007XFT, All Rights Reserved

ISO9001
ISO13485

Manufactured for: WNL Products 330 Hopping Brook Park
Holliston, MA 01746 (800)884-9829

www.wnlproducts.com

- Thank you for purchasing WL 220 Practi-Trainer
- Please read the instruction manual carefully and thoroughly before operating the machine
- Keep manual available for future reference

Ver: XFT120020708

Product Introduction

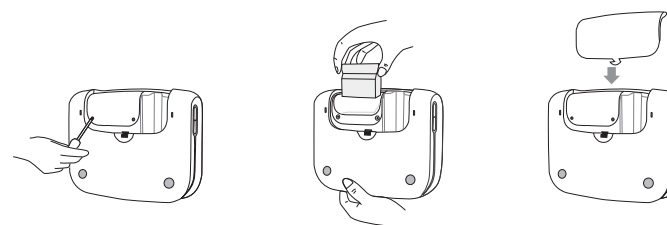
The WL220 Practi-Trainer is designed to effectively train anyone in the use of an Automated External Defibrillator (AED). This universal AED training unit provides a variety of pre-programmed scenarios to help trainees become familiar with the functions of an AED and to allow them to demonstrate the basic skills necessary to use an AED in an emergency situation.

The WL220 provides simulated shock delivery. To ensure safety during training, it has no high voltage capabilities and cannot deliver a shock.

NOTE: Before using a real, clinical AED, the responder should complete an appropriate training program and have demonstrated proficiency in Basic Life Support skills.

Set Language

This unit generally ships with a language/scenario card already installed. If not, or if purchasing a new language or product update, this diagram shows how to install the plug-in module. You will need a small Philips head screwdriver to open the module cover.

This plug-in design makes it easy to add different languages and to update your unit when new ECC guidelines are issued.

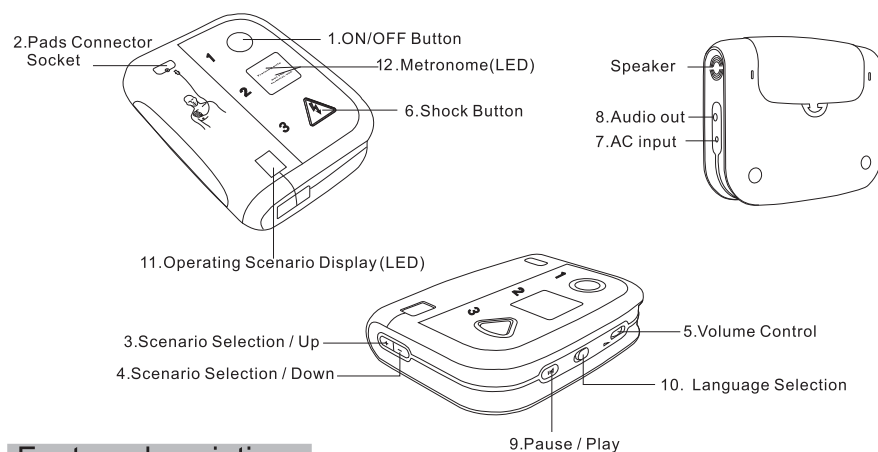
Directions for Use

Overview

Product Content & Feature Review

The WL220 includes:

Main body, Remote Control, Plug-in module, 1 set each of Adult and Child training pads, Wire connectors for adult and child and Storage Bag.

Feature descriptions

1. On/Off button

Press once to turn unit on and press again to turn unit off.

2. Pads Connector Socket

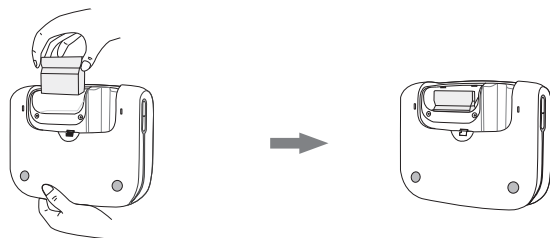
Insert the training pads connector into this socket. The LED flashes to indicate location as voice prompts to plug it in. Once properly connected, the LED is covered.

3/4. Scenario Selection


There are pre-programmed scenarios. Press these buttons to scroll up and down to change the scenarios.

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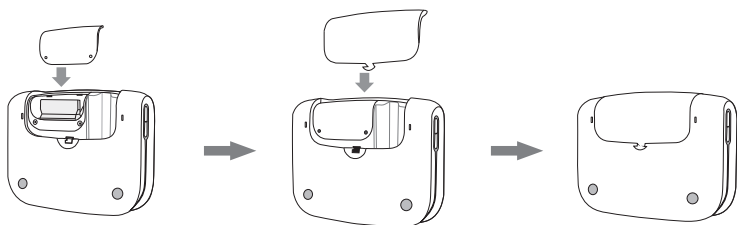
Directions for Use

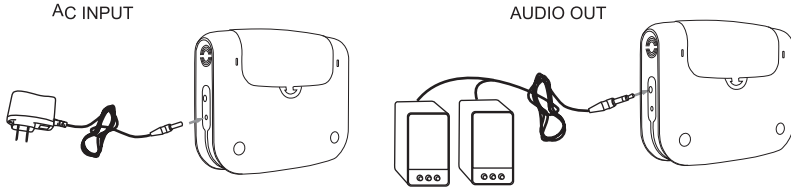5. Volume Control
Controls the volume of the voice prompts during use.
6. Shock Button
Press to “deliver” simulated shock. The button flashes when the unit is ready to deliver the simulated shock and the voice prompt indicates to press this button.
7. AC input
This interface connects the AC adapter(sold seperately). When not using batteries to power the unit, plug in the AC adapter for power. Adapter is for use with AC4.5V/300mA. DO NOT use batteries when adapter is plugged in.
8. Audio Out
External Speakers can be connected to the Practi-Trainer with a standard audio jack.
- 
- The diagram shows two connection scenarios. On the left, labeled 'AC INPUT', a power adapter is plugged into the side of the device. On the right, labeled 'AUDIO OUT', two external speakers are connected to the side of the device via audio cables.
9. Pause/Play
This button can temporarily stop all operations of the unit. Press once to pause, press again to continue.
10. Language Selection
The WL220 is designed with bilingual scenarios. The user can change the language of the scenario prompts using this toggle switch. Language selection is dependent upon the module that was purchased and installed into the unit.
11. Displays which scenario(number)which is currently selected and active.
12. Metronome
Flashing LED and audible “beep” count out the proper CPR compression rate of 100 per minute.

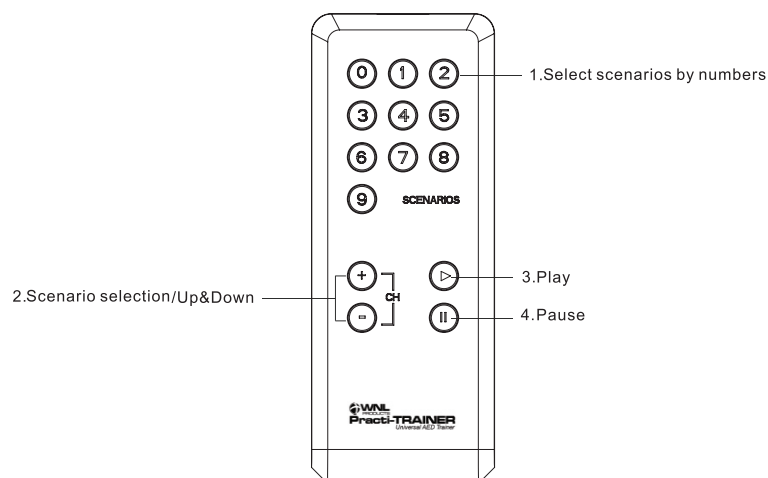
Directions for Use

Automatic Shut Off

1. When using any scenario, if no operation is indicated after 5 cycles, the unit will turn off automatically.
2. If the remote control is being used, the AED Trainer will turn off after 5 cycles if no functions are selected.
3. If low power is detected during use with batteries, the unit will turn off in 10 seconds. Plug in adapter if new batteries are not available.
4. If the unit is paused, with no other activity, it will turn off after 8 minutes.

Remote Control


Remote control button functions are described below:

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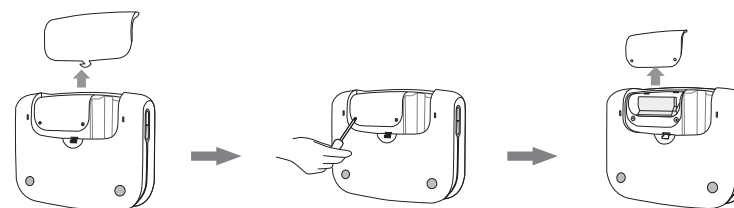
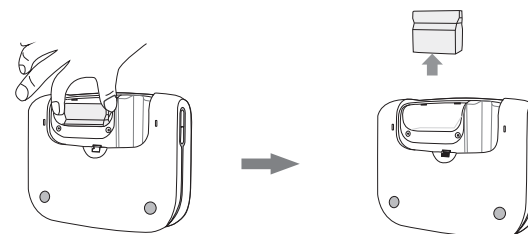
Directions for Use

1. Scenario selection
press the number of the scenario you want to use.
2. Scenario selection up/down
scroll through the scenarios.
3. Play
press to resume activity after unit was paused.
4. Pause
press to freeze the scenario. LED window shows "//".

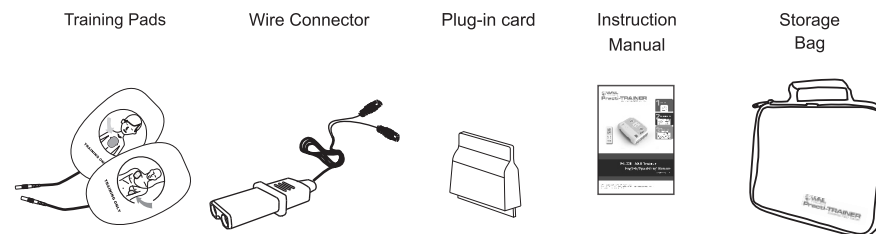
Other Components


Directions for Use

Battery Installation

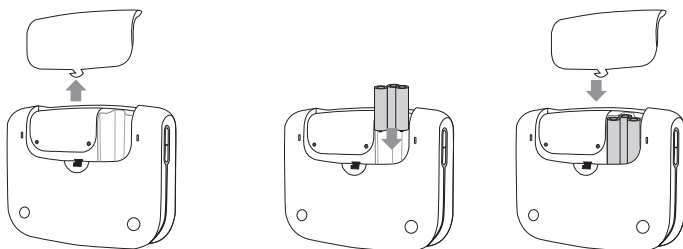1. Open the battery compartment in the back of the unit by pushing in the direction of the arrow.
2. Install 3 AA batteries noting the proper polarity. Please assure you are using fresh batteries for best performance. (Note: a clinical AED unit will not require this type of battery installation)
3. Reposition the battery compartment cover.
4. The unit will indicate low battery with a voice prompt. When this is heard, remove old batteries and replace them or plug in the AC adapter.

Training Pads

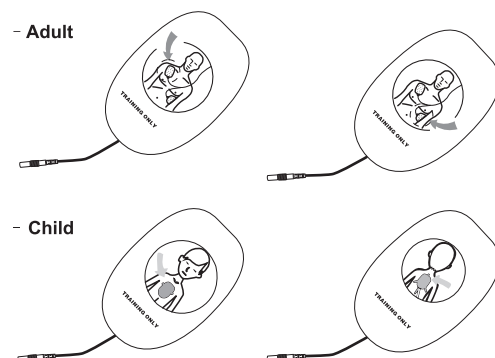Note: The training pads are for training purposes only and will not function with a clinical AED device.

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
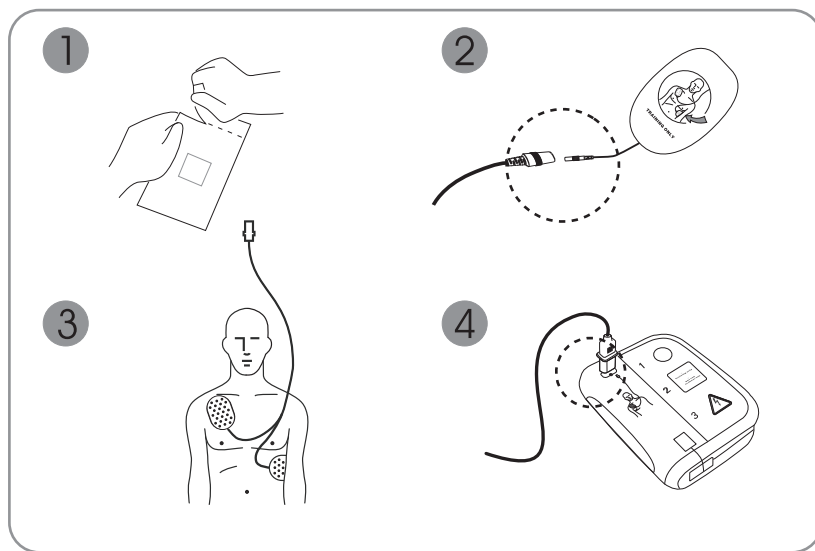
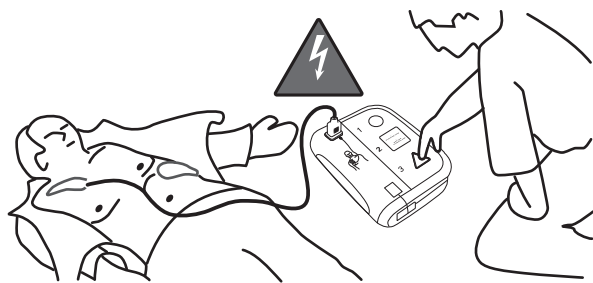
Directions for Use


Operation Instructions

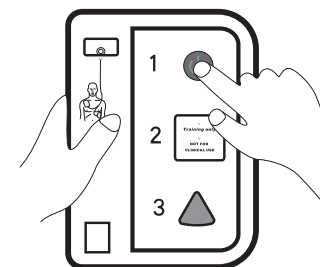- 1) Press the power button to turn the unit on. Then select the desired training scenario. The number of the scenario will be displayed in the LED window. (Note: the last setting will be stored when the unit is shut off)

- 2) Remove the adult or child training pads from their package and connect them to the appropriate wire harness – yellow connector for the adults and pink for child.
- 3) Properly position the pads on the manikin's ("patient's") chest – as shown in the diagram.
- 4) Plug the connector into the socket on the training unit when prompted to do so.

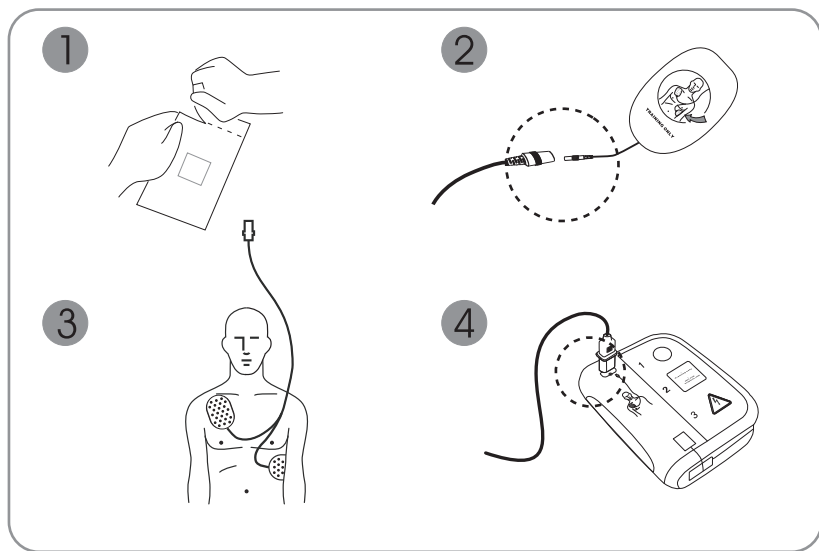
Directions for Use

Note:

- Do not stretch the connection cord or bend it excessively.
- Always hold the connector itself when pulling it out of the socket.
- Use care when removing pads from manikins– do not pull on the cord and grasp the pad itself firmly.

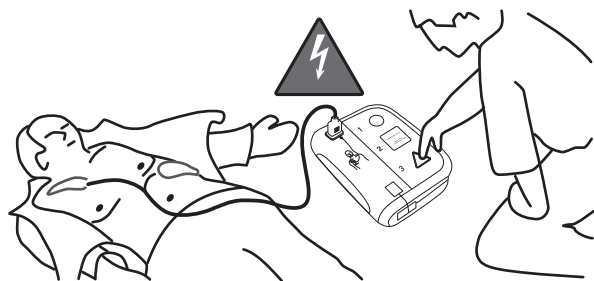
5) Finally, follow the voice prompts for your selected scenario

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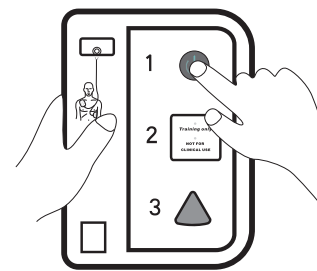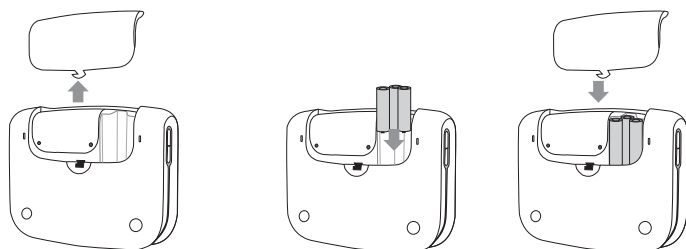
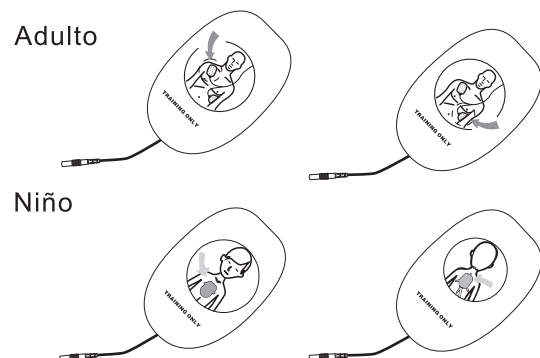
Storage Information

Pads Maintenance

- Keep gel side of pads clean, dust free and covered with original packaging when not in use.
- If pads loose some adhesion properties, they can be rinsed gently with water and left to dry – adhesive side up.
- Never wipe the gel adhesive with paper towels or cloth.
- Do not scrape the pads.
- With proper care, the pads should last for several training sessions.
- Do not wash pads frequently and never use detergent or hot water.
- Do not store pads in direct sunlight or in an area with high temperatures or humidity.

Main Body

- After turning the unit off, make sure to pull out the connector and store everything properly.
- If the unit is dirty, use a soft cloth with water and a neutral detergent, Wring the cleaning cloth well – do not immerse in water or any liquid.
- If necessary, clean with a cloth dampened with alcohol (75% concentration max.)
- Do not use abrasive cleaners.

Storage Precautions

- Do not store near spirits or gasoline.
- Do not store in direct sunlight or where the unit will be exposed to high temperatures, moisture, excessive dust or corrosive materials.
- Store all parts in the bag provided.

*Failure to comply with these instructions may void the warranty.

Directions for Use

8 Pre-programmed Scenarios


8 preprogrammed scenarios in WL 220 AED Trainer.

Scenarios	Description	Operation
1	Ventricular Fibrillation- With single-shock conversion	Shockable rhythm 1 shock CPR Non-shockable rhythm CPR (then repeats)
2	Ventricular Fibrillation- multiple shocks required	Shockable rhythm 3 shocks CPR between shocks Non-shockable rhythm CPR
3	Troubleshooting Skills- Defibrillation pads	Poor pad connection (10 sec.) Shockable rhythm 1 shock CPR Non-shockable rhythm CPR
4	Ventricular Fibrillation- two shocks required	Shockable rhythm 2 shocks CPR between shocks Non-shockable rhythm CPR (then repeats)
5	Non-Shockable Rhythm	Non-shockable rhythm throughout 3x w/ CPR in between
6	Troubleshooting Skills- Motion	Detect motion Shockable rhythm CPR Non-shockable rhythm CPR (then repeats)
7	Troubleshooting Skills – motion after initial shock	Shockable rhythm 1 shock CPR Detect motion Non-shockable rhythm
8	Ventricular Fibrillation – conversion then refribrillation	Shockable rhythm 1 shock CPR Non-shockable rhythm CPR Shockable rhythm 1 shock CPR

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
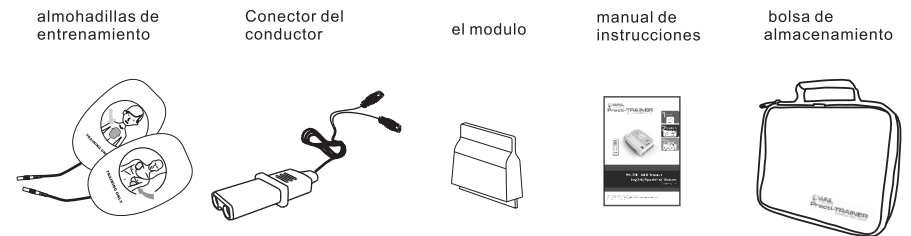
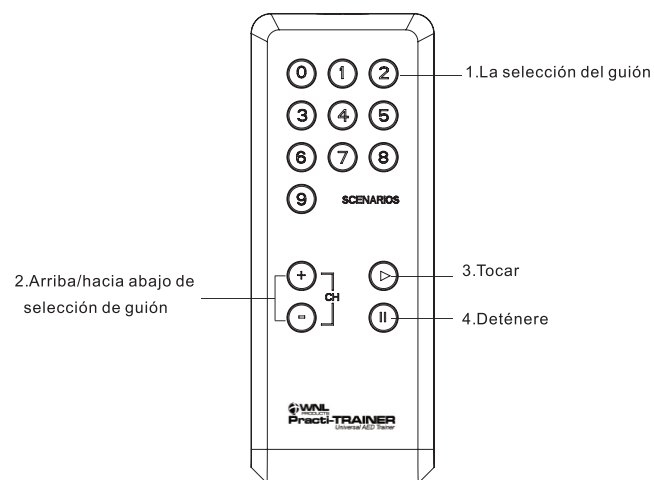
Directions for Use

How to Replace the Language/Scenario Module

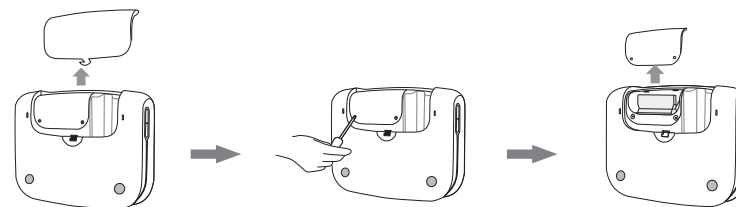1. Make sure the unit is turned off. Remove the battery cover on the back of the unit. Unscrew (small Philips head needed) the 2 screws on the compartment cover to access the plug-in module.

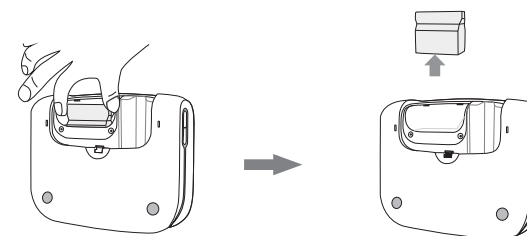
2. Grab the plug-in module on the sides and carefully pull straight up.


Directions for Use

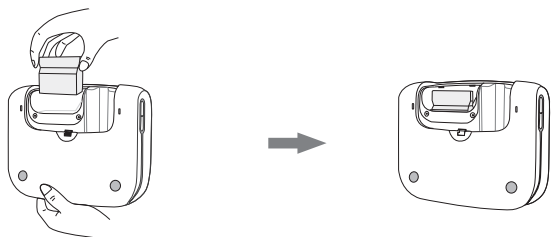3. Take your new module, check for proper orientation (printed/stamped side up) and push it straight down into the connector.

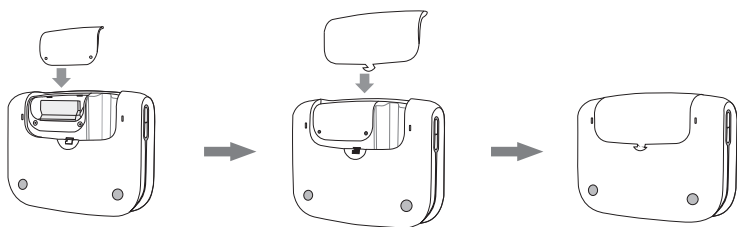
4. Replace the compartment cover and the main battery cover. Turn unit on to read the new module.

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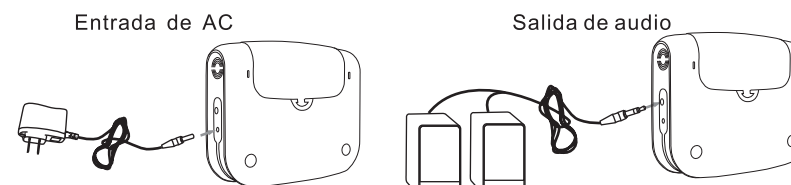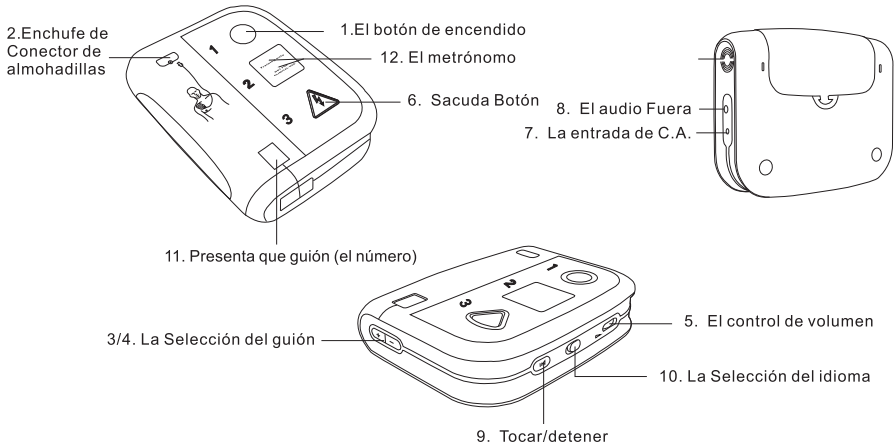
Directions for Use

Specification and Product Contents

Model No: WL220 series
Name: AED Practi-TRAINER with replaceable Language/Scenario module
Power Supply: AC 4.5V / 3 AA batteries
Product Dimensions: 190mm x 150mm x 46mm
Static Current: <200uA
Maximum Current: <300mA

Remote Control:
Power: 2 AAA batteries
Static Current: <5uA
Maximum Current: <10mA

Product Contents:

NO.	Contents	Quantity	Unit	Remark
1	Main Body	1	pc	
2	Plug-in Card	1	pc	
3	Remote Control	1	pc	
4	Wire Connector	2	pcs	1 yellow connector, 1 pink connector
5	Adult Pads	1	pair	These pads can be used repeatedly
6	Child Pads	1	pair	These pads can be used repeatedly
7	Instruction Manual	1	pc	
8	Storage Bag	1	pc	Nylon bag

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